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No.365, Wuzhou Road, Yuhang Economic Development Zone,
Hangzhou city, 311100, Zhejiang, China

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reer GmbH • Mühlstr. 41 • 71229 Leonberg • Germany • www.reer.de

Enter your data in the app in settings „My cycle“.

IMPORTANT! THE FOLLOWING INSTRUCTIONS MUST BE READ CAREFULLY AND RETAINED FOR FUTURE REFERENCE KEEP INSTRUCTIONS SAFE!

Legend

The following symbols are used in the user manual, on the packaging and on the type plate of the device and the accessory:

	Caution
	Permitted storage and transport temperature and air humidity
	Permitted operating temperature and air humidity
	Manufacturer
	Observe the user manual
	Application part type BF
	Disposal according to the Waste Electrical and Electronic Equipment EC directive
	Batch designation
	Protection against entry of solid foreign bodies with a diameter of >12.5 mm. Protected from temporary diving.
	CE mark of the designated authority (TÜV Rheinland)

Scope of delivery

- Thermometer
- Operation manual
- Protective case
- 1x 3V battery CR2032

Safety instructions

- The thermometer is intended exclusively to measure human body temperature.
- The thermometer is designed to only be used at the measurement points on the human body that are specified in the user manual.
- Check the unit for visible damage before each use. If you detect any damage, do not use the unit.
- The thermometer is intended for private use.
- Children should not use the device without adult supervision. Medical products are not toys.
- Children under 12 should not measure their own body temperature without supervision.
- This thermometer contains small parts that can be swallowed or present a choking hazard for children. Always keep the thermometer out of the reach of children!
- Do not open the device, with the exception of the cover on the battery compartment. No modifications may be made to the device. If a device does not work correctly, please contact your dealer.
- Keep the battery out of the reach of children as there is a risk of swallowing.
- The thermometer contains sensitive electronic components. Protect it against impacts, bending, high temperatures or direct sunlight.
- The minimum measurement time until the sound signal must always be observed.
- In the event of unusual temperature readings, immediately contact your GP.
- Please note that portable and mobile HF communication equipment can affect this device. You can obtain more detailed information at the customer service address specified.
- Disinfect the thermometer after every use.
- Do not take readings when the thermometer is wet, as this could lead to imprecise measurement values.
- The thermometer is NOT suitable for contraceptive purposes.

Before first use

Download the „reer basal thermometer“ app for free in the App Store or Google Play Store.

Activate Bluetooth on your smartphone and go to „Settings“ and „Device connections“ in the app. Tap on „Device connections“ and switch on your thermometer with the on/off button next to the LCD display. The display will briefly show with a beep, followed by the temperature last saved. This process takes about 3 seconds. Then select the device ID via the app and connect the devices. The app is synchronised with the thermometer while they are connected.

Notes

- When the temperature measured is less than 32 °C, the LCD display will show „Lo“. When the temperature measured is higher than 42,99 °C, the LCD display will show „Hi“.
- If “---“ appears in the LCD display, the thermometer is waiting for a Bluetooth connection. You have 30 seconds to connect your thermometer. Otherwise, the Bluetooth connection will be skipped and normal temperature measurement will begin.

Measurement:

Take your basal body temperature daily immediately after waking up in the morning and before getting up. Always take the measurement identically (time and place). Observe the minimum measuring time of three minutes until the beep. The reading will also be continued after the beep.

Document the temperature pattern over at least 2 menstrual cycles and start at the beginning of your menstrual period. Measurement accuracy may be influenced by external factors, such as alcohol/medication consumption, a lack of sleep, or stress.

Measurement points:

In the mouth cavity (oral)

Place the thermometer under your tongue, to the left or right by the base of the tongue.

Close your mouth and breathe evenly through your nose to prevent the reading being affected by air breathed in or out.

Taking the reading in another location, such as under the arm, results in inaccurate values and is not recommended.

The measured values are automatically transmitted to the app. The temperature recording serves as a guide. For greater accuracy, the consistency of the cervical mucus can be observed. The Fertility depends on various factors, such as age, cycle length, and other medical conditions. If in doubt, consult your doctor.

By taking daily measurements, a temperature curve is created in your app, which you can use to monitor your cycle precisely.

Based on your entered data under „My cycle“, the different cycle phases are displayed in colour:

- green stands for your fertile days
- the blue dot marks ovulation
- pink is the time during menstruation
- orange indicates your infertile days

Changing the batteries

When the symbol appears in the display, the battery has run out and a battery change is required. Replace the battery with one of the same type. For this purpose, proceed as follows:

Unlock the cover of the battery compartment (with a coin, for instance) through a quarter turn to the right. Remove the old battery and insert the new battery (type CR2032).

Please take note of the installation position. The + sign on the battery must point up. Then close the cover on the case again. Batteries can be obtained in specialist watch or electrical shops.

Used batteries should not be placed in household waste. You are legally obliged to dispose of the batteries separately. Dispose of them via your specialist electrical retailer or your local recycling point.

Notes on handling batteries

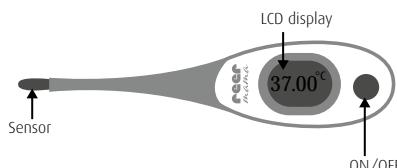
- Remove the battery from the unit if it is to be out of use for a longer period.
- If a battery cell leaks out onto your skin or eyes, wash the affected area with water and seek medical help.
- If a battery has leaked, put safety gloves on and clean the battery compartment with a dry cloth.
- Protect batteries from excess heat

Risk of swallowing!

Small children can swallow batteries and choke on them. Keep batteries away from small children!

Metrological control

As a rule, checking the thermometer's precision is not required. For devices used in Germany for medical practice, the Medical Products Operator Ordinance of 29 June 1998 (in the version announced on 21 August 2002, last amended by Article 4 of the law of 29 July 2009) requires metrological control after 2 years. Gauging offices provide paid metrological inspections.

Cleaning and disinfection

- Disinfect the thermometer and the measuring tip with alcohol (75% isopropyl alcohol). All pharmacies sell alcohol cleaning pads from various manufacturers, with which you can completely wipe the device.
- Wipe the thermometer dry with a soft cloth.
- DO NOT BOIL OR WASH IN DISHWASHER. This will negatively affect the functional capacity of the thermometer and will void the warranty.
- Do not clean the device with chemical thinners and solvents.
- Store the thermometer in a dry, dust-free, clean place away from direct sunlight.

Microbial contamination

Because complete disinfection is often not possible in household use, it is highly recommended to provide an individual thermometer to each patient.

Disposal instructions

Batteries must be disposed of appropriately. Appropriate bins for battery disposal are available for this purpose at retail shops that sell batteries, and at communal collection points. After the end of its service life, please dispose of the device correctly through a community collection point for waste electrical products.

Legal basis for marketing the device in the EU:

The manufacturer was certified according to the guidelines of 93/42/EEC for medical products. This device complies with the EN60601-1-2 European standard on electromagnetic compatibility and meets the requirements of EN 12470-3.

Warranty

Please retain your receipt, the instruction manual and product number. Warranty claims or claims under other promises of guarantee must be processed by the dealer. The statutory warranty period is 2 years from the date of purchase. Liability for defects does not include damage caused by inappropriate handling. The purchase receipt and a description of the defect are required for warranty claims.

The technical specification, instructions and characteristics of the product described herein have been compiled to the best of our knowledge and belief, and are correct at the time of printing. Any instructions provided by reer should be correct and reliable. Nevertheless, reer does not accept liability for potential errors in this document and reserves the right to make changes to the product design and/or specifications without prior notice.

Technical data

Type:	reer mama Basal thermometer
Item number:	31020 / DMT-4735b
Measuring range:	32,00°C to 42,99°C (89,60°F-109,3°F)
Measuring accuracy:	±0,05°C (±0,1°F) at a temperature between 35,00°C~38,00°C (95,00°F~100,40°F) ±0,1°C (±0,2°F) in a water bath T<35,00°C (95,00°F) or T>38,00°C (100,40°F)
Measuring units:	Celsius (°C)
Display:	4½-digit liquid-crystal display (LCD display)
Battery type:	1x CR2032 (included)
Net weight:	approx. 20g
Ambient temperature	
Operating	+5 °C to +40 °C, at 15% - 95 % relative air humidity
Storage/Transport	-20 °C to +55 °C , at 15 % - 95 % relative air humidity
Measuring time	3 minutes
Expected lifetime of thermometer	3 years
Battery life	Approx. 300 measurements (with a 3-minute measuring time)

EC REP Company Address

Shanghai International Holding Corp. GmbH (Europe)
Eiffestrasse 80, 20537 Hamburg, Germany

JOYTECH HEALTHCARE Co., Ltd.

No.365, Wuzhou Road, Yuhang Economic Development Zone,
Hangzhou city, 311100, Zhejiang, China

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