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Wellness-Kühlsocken // Art.Nr.: 31041

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## GB Instruction manual

Wellness cooling socks // Item no.: 31041

**IMPORTANT! THE FOLLOWING INSTRUCTIONS MUST BE READ CAREFULLY AND RETAINED FOR FUTURE REFERENCE KEEP INSTRUCTIONS SAFE!**

### General safety instructions:

- Please only use the item for its intended purpose.
- Discontinue use if there are signs of defects.
- Please follow the heating instructions without fail, as otherwise, there is a risk of the gel pack bursting.
- Only use the socks for private use. They are not intended for commercial, medical, or therapeutic use. They are not a substitute for treatment by a doctor. If you have any health concerns, consult your doctor before using the item.
- People with circulatory problems and diabetics should consult a doctor before use.
- Keep the item out of the reach of children.
- Do not use if you have serious or chronic illnesses or irritated skin.
- Do not run in the socks – there is a risk of slipping!
- Remove gel packs from the socks before heating or cooling.
- Never place gel packs directly on the skin.
- Allow the gel packs to cool down at room temperature before (re-)heating.

### Action to be taken in the event of direct contact with gel:

If the contents come into contact with the skin or eyes, rinse immediately with water. The gel is not toxic.

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## Instructions for use:

### For warm use:

- Microwave: Heat the gel packs for about 20 seconds in a 700 W microwave.

If the gel pack is not warm enough, heat it once again for 10 seconds. Knead the gel pack, turn it over, and heat it a second time to distribute the heat evenly. Note: NEVER heat the gel pack too much.

- Boiling water: The gel pack can also be immersed in boiling water for 7-10 minutes. Use tongs to remove the pack from the water.

### For cold use:

Place the gel pack in the fridge for around 1.5-2 hours. For a stronger effect, the gel packs can also be cooled in the freezer for a maximum of 8 hours. Your legs can be elevated to increase the effect.

## Care instructions:

Socks: 100% polyester, machine washable at 30 °C. Allow to dry in the air. No bleaching, no tumble drying, no ironing, and no chemical cleaning.

Gel pack: Wipe with a damp cloth.

Please retain your receipt, the user manual and item number. Warranty claims or claims from promises of guarantee must be processed by the dealer.

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