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**GB Instruction manual**

Grape seed nursing aid pillow // Item no.: 31092

**IMPORTANT! THE FOLLOWING INSTRUCTIONS MUST BE READ CAREFULLY AND RETAINED FOR FUTURE REFERENCE KEEP INSTRUCTIONS SAFE!****General safety instructions:**

- Please only use the item for its intended purpose.
- Discontinue use if there are signs of defects.
- Keep the item out of the reach of children. Item contains small parts that may be swallowed. Danger of choking!
- Do not use if you have serious or chronic illnesses or irritated skin.
- Remove pad from the outer cover before heating and cooling.
- Caution: Do not heat pad for longer than the time specified. Fire hazard!
- Always use with the outer cover and do not place directly on the skin.
- Allow to cool down at room temperature before (re-)heating.
- Store pad in a clean, dry and airy environment.
- If in doubt, speak to your doctor or midwife.

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**Instructions for use:****Before nursing / warm use:**

- To provide relief from lumps or engorgement and stimulate milk flow, heat can be applied to the affected place for a few minutes before nursing or before breast emptying.
- Ensure that the filling is evenly distributed on the inside to avoid burning!
- Always place half a cup of water, for example, in the microwave or oven while heating. Ensure that the container is fire-proof.
- To heat in the microwave: Heat for max. 15 seconds in an approx. 750 W microwave.
- Ensure that the plate can turn freely and is free from residue.
- To heat in the oven: Heat for max. 6 minutes at approx. 110 °C.

**After nursing / cold use:**

- After nursing or emptying the breast, cold applications are recommended. The cold application should last max. 20-30 minutes.

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- Place it in the freezer in a well-sealed plastic bag for around 1-2 hours. Wait for 1-2 minutes before placing the pad with the outer cover on your body.

**Cleaning:**

- The grape seed filling cannot be washed!
- Inner layer: 100% cotton, non-washable to avoid mould forming
- Outer layer: 100% cotton, machine washable up to 30 °C. Allow to dry well in the air. No bleaching, no tumble drying, no ironing and no chemical cleaning.

Please retain your receipt, the user manual and item number. Warranty claims or claims from promises of guarantee must

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